

ZH05C Air Massage Mattress With Lifting Function
Operation instructions

SAVE THESE INSTRUCTIONS



©For reference only

Thanks for purchasing this product. Please read this manual instruction carefully in order to know how to use this product correctly before operation. Please pay more attention to some important safety warning and keep this manual book properly for future use.

Note: Our Company reserves the right to revise the design and description of this product without any further notice. Please refer to the actual color of this product.

CONTENTS

Important Safety Warning	2
Function	2
Name And Function Of Components	3
Start To Massage	4
The Adjustment Of The Angle Between Backrest And Calf-rest Of Bed Base	5
Restoring When Power Cut	6
Language Setting	6
Set Remote Communication Address Code	7
Common Phenomenon Disposal	9
Product Specification	9

⚠ IMPORTANT SAFETY WARNING

- This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- For massage appliance with power cord, if the power cord is damaged, it must be replaced by the manufacture, the maintain department or similar professional person in order to avoid danger.
- For massage appliance with adapter, if the adapter is damaged, it must be replaced by the manufacture, the maintain department or similar professional person in order to avoid danger.
- For the massage appliance with heating function, please note the appliance has a heated surface, person insensitive to heat must be careful when using it.


FUNCTION

Air massage mattress with lifting function


- This mattress is composed of modified polyurethane High-flexible foam, using natural environmental protection material as surface fabric, with thermal insulation character such as venting and environment protection.
- The unique design of product can evenly contour to body curve and distribute body weight. There are air pressure massage, airbags inside the mattress corresponding to the body of the neck, shoulder, back & waist, buttocks, foot. The leg is equipped with a vibration massage ball. When start massage function, people lie on the mattress enjoy air pressure massage and various massage methods. It will improve blood circulation, ensure deep stage sleep, prevent or reduce tension, numbness and sore of your muscles, and relieve snoring and insomnia.
- Inside with 25 pieces high magnets which can carry on magneto therapy to every acupoint of human body .
- With 24V dynamo external power make the mattress working , safety and low consumption , with remote wireless controller , easy for handing . The back and legs of the mattress were controlled by air pressure, which can adjust the angle of the back and the height of the legs, to make human get a natural posture. When power off, turn the air escape cock to restore to flat state.

FUNCTION

25 pieces high magnetic distribution inside foam

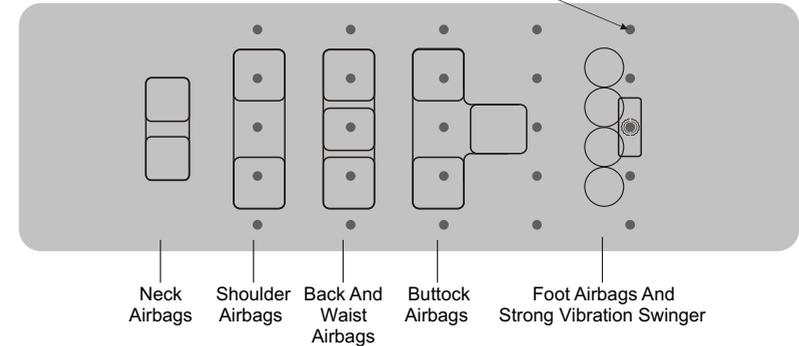

NAME AND FUNCTION OF COMPONENTS

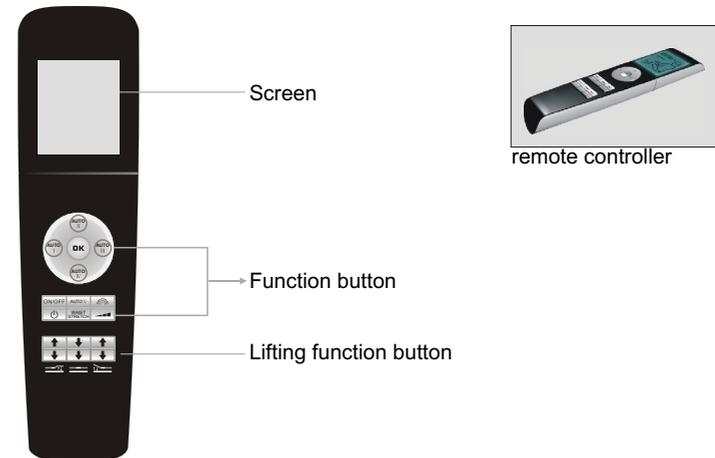

START TO MESSAGE

1. Plug in the machine 、 instruction manual

Step1


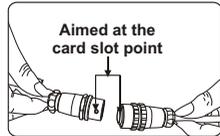


Figure : Connecting the power of the mattress

Step2

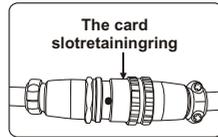

After docking, the retaining ring must be across the card slot point (making a sound) then fixed interface.

Step3




Figure : Connecting the power configuration of the mattress

Attention

- 1.Connecting the power, two sides of power line docking port should arrow aligned, slight enforce, or hard plug will damage the inside needle.
- 2.Be sure not to switch adaptor power unless you finish all the steps above, otherwise you will cause damage to the machine

2.Message

Press the button “” on the handheld remote controller, you can see the communication testing interface and the testing function start, if the handheld remote controller communicate well with the controller of the massage bed, and the handheld remote controller will display the test was successful and change to massage interface (and the default model is automatic 1;Vibration massage on ,with 2 levels), or else display the test was failed and presents the message

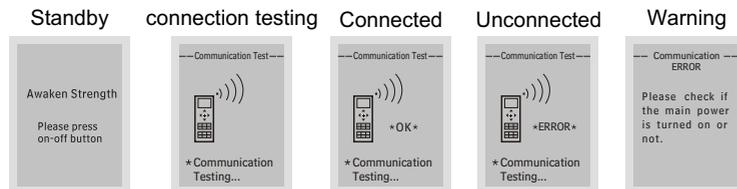

START TO MESSAGE

Name	Button	Message Content	Message Features	Display
Massage model		Auto 1	Message focus on whole body air pressure, loosens the limbs (Advice lay on bed and flat on back massage)	 Auto I
		Auto 2	This model is to remove the neck massage function in Auto 1. (Advice lay on bed and flat on back)	 Auto II
		Auto 3	Message focus on neck and shoulder, buttock, improve the fatigue symptoms. (Advice lay on bed)	 Auto III
		Auto 4	This model is to remove the neck massage function in Auto 3. (Advice lay on bed and flat on back massage)	 Auto IV
		Auto 5	Message focus on belly, promote the gastrointestinal function. (Advice lying face up on a bed)	 Auto V
		Waist Stretch	Message focus on push waist to alleviate the pain on the waist.(Advice lay on bed and flat on back massage)	 Waist Stretch
Vibration model		vibration switch	Switch on, foot Vibration massage.	 Vibration massage
		Vibration intensity	With 3 levels adjustable	

THE ADJUSTMENT OF THE ANGLE BETWEEN BACKREST AND CALF-REST OF BED BASE


1) Adjust the degree of the backrest

- Back up button: press this button when turn on the power, the back will up gradually, with dripping sound at the same time. (During two minutes this button not touched or mattress go to the top position, the power will be off automatically). Press this button again to stop it.
- Back down button: press this button when turn on the power, the back will drop gradually, with dripping sound at the same time. (If it was no reaction within two minutes, then you can press this button again to turn it off). Press this button again to stop it.

THE ADJUSTMENT OF THE ANGLE BETWEEN BACKREST AND CALF-REST OF BED BASE


2) Adjust the degree of the calf rest

- Back up button: press this button when turn on the power, the legs will up gradually, with dripping sound at the same time. (During two minutes this button not touched or mattress go to the top position, the power will be off automatically.). Press this button again to stop it.
- Back down button: press this button when turn on the power, the legs will drop gradually, with dripping sound at the same time. (If it was no reaction within two minutes, then you can press this button again to turn it off). Press this button again to stop it

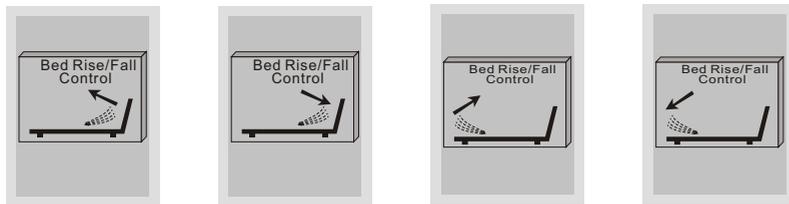


3) Adjust the angle of the backrest and calf rest together

- Lay down and press the button at the massage condition, the backrest and calf rest will drop together, with dripping sound at the same time. (If it was no reaction within two minutes, then you can press this button again to turn it off). Press this button again to stop it.

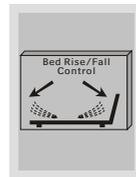

RESTORING WHEN POWER CUT


When the power cut, and the mattress at the lifting state, press the recovery button to make the mattress restore to flat state.

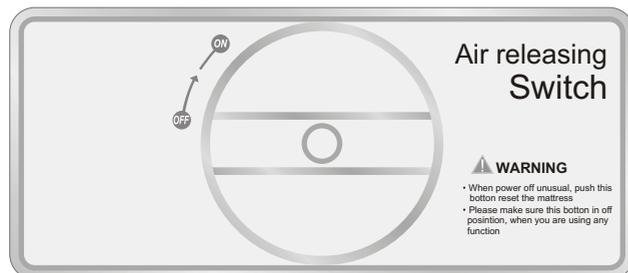

LANGUAGE SETTING

1. Into Chinese/ English display optional interface

In standby state, press “Auto III” button and the back lift button “↑” at the same time, then release, into Chinese/ English display optional interface.

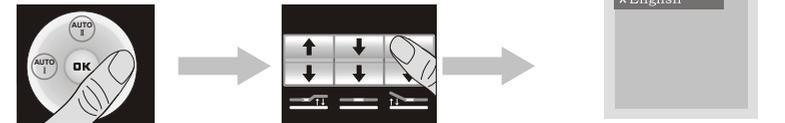

2. Choose the language you need

Pressure the function button “AUTO I” (rise) and “AUTO II” (down) to choose, the pressure “OK” button, it will be ok.


SET REMOTE COMMUNICATION ADDRESS CODE

Under the below situations, it needs to reset IP:

- 1) Change the hand held controller.
- 2) The hand held can't start the corresponding function, or always showing communication error.

I .Steps for setting communication password of mattress.


1. Press the address restoration switch of the inside steam resource box of the mattress for 3 seconds, when there is ringing sound of [BI] , [BI] , it shows the communication address of the main controller is restored. The schematic diagram of the address restoration switch on the steam resource box inside the mattress.

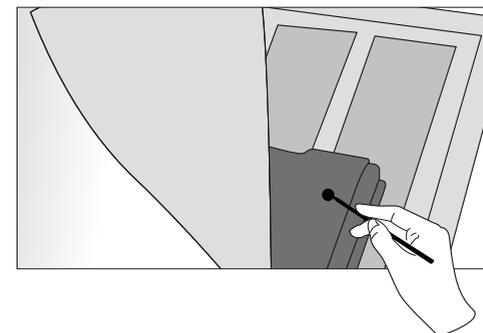

SET REMOTE COMMUNICATION ADDRESS CODE

2. Into the address setting interface


With the standby state of the handheld remote controller, press the “Auto II” button and the “AUTO V” button at the same time, then release, in to the address setting interface (the two buttons pressed at the same time)

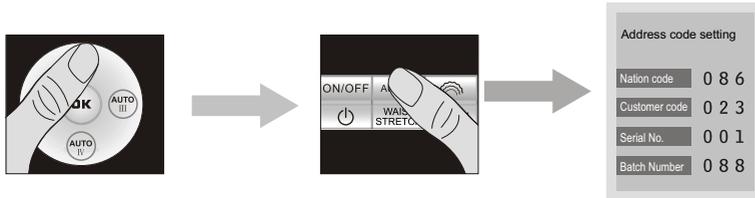

3. Set communication address value through Navi and Digital Keys, totally 4

address values, setting range of each address value is among 0 to 255. (Schematic diagram of remote controller digital keys)

Setting method: the “* ” sign in the communication address setting interface represents the current data bits to be set, change the data bits through Navi Key, and set the value of current data bits through digital key.

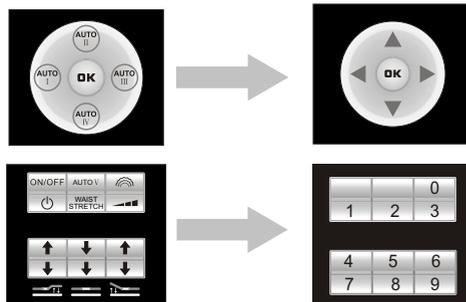

4. After set communication address value, press [OK] key to confirm, the communication address setting is successful if heard once “bee” hoot and the interface is turned to standby interface, the communication address setting is failed if heard twice “Bee” hoots, re-press [OK] key to confirm or return to the 1st step to reset.

COMMON PHENOMENON DISPOSAL

Phenomenon	Potential cause	Disposal method
The product doesn't work when power on	Unwell contact of both lifting mechanism plugs	Inspect the plug of lifting mechanism and driven box to make them contact
	Unwell contact between plug and socket	Inspect power of socket to ensure plug and socket contact well
When pressing the lifting key, no up and down changes for the bed frame	Whether the power cut and air release switch are off	Rotate the power cut and air release switch to keep them off
Machine suddenly stop working	Power on or not	Inspect and recover

PRODUCT SPECIFICATION

Model: ZH05C

Name: Air massage mattress with lifting function

Rated Voltage: 110-120V~ 60Hz
220-240V~ 50Hz/60Hz

Rated Power Input: 85W

Safety Structure: